

# DISINFECTION

The following items of equipment manufactured by ThermoSecure medical equipment GmbH,

**TC50K, TC50KHD, ThermoCare TC 40 / 45, ThermoCare MX 45 / 55, ThermoCare MX 75, ThermoBox TB 3, TB10, TB 30, TransCooler** are disinfected and cleaned as follows:

## 1.) Manufacturer & Product

Disinfectant:

“**Microbac forte 5 %**” from Bode Chemie, Hamburg or

“**Pursept-FD**” (from Merz-Hygiene).

There is no danger of damaging the materials used when these products are applied as recommended by the manufacturer.

Gloves should be worn when handling the products.

## 2.) Components to be disinfected :

**Regularly:**

All housing surfaces, the inner surfaces/cooling area, inside lid.

The keyboard must be cleaned with a damp cloth only and **liquids must not** penetrate the equipment.

**ALWAYS disconnect equipment from the POWER SOURCE when disinfecting!**

**Basic cleaning:**

Undo the 4 screws of the fan-grid fastening in the inner space so that the surfaces behind the fan can be disinfected.

Attention must be paid that only instructed technical staff conducts this!

The cabling must not be damaged or twisted, as otherwise the function of the fans, heating or sensors can be affected. The warranty expires immediately if this is ignored.

## 3.) Application

No rough or sharp items/aids may be used for cleaning. Work with a soft cloth only, better still is to spray only. Be particularly careful during the **basic cleaning** and make sure that cables and connections are not damaged!

There must not be any “puddles” after completing disinfection. Residue must be mopped up. Particularly during the **basic cleaning** care must be taken that the electrical connections do not stay damp!

The whole unit must be well ventilated and completely dry before taking it into operation again.